




## 2. Product Use

1


Click the power switch located on the right side of the product to turn the power on.

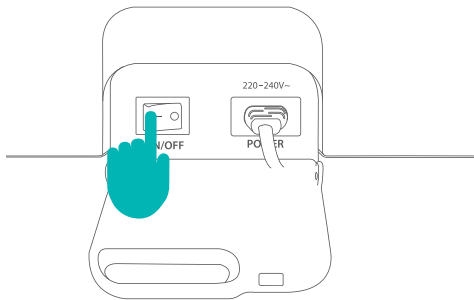

2


Once the LED light on the Start/ Pause button has lit up, press it to start the program.

(Default: auto mode)

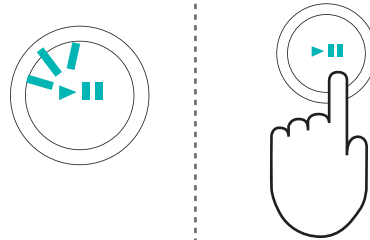

3


Use the ON/OFF button to control the sound.  
(Applicable to both Auto/Manual modes)

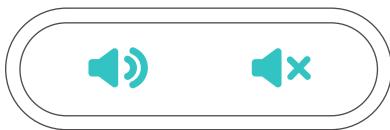

4

Turn the temperature setting dial from 30~60°C to set the projector's temperature.  
(Applicable to both Auto/Manual modes)

Control the power using the ON/OFF button on the external projectors.

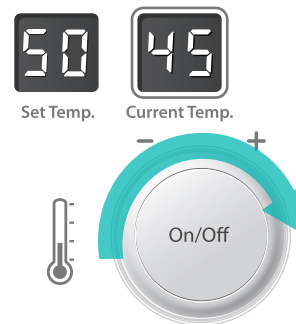

5

Pausing the Product

1. Press the Start/Pause button to pause at anytime during usage.
  2. When in the Pause mode, the time indicator will flicker on and off.
  3. Other buttons will not be functional while in Pause mode.
  4. Press the Pause button to resume the program.
- The product will turn off automatically after 10 minutes in Pause mode.