





3. Manual Mode

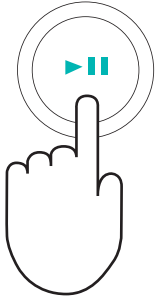


- 1** Once the product has been switched on, press the Start/Pause button.




- 2** Manual Mode operates from 5 ~ 120 minutes. The product will shut down automatically once the set time has passed.

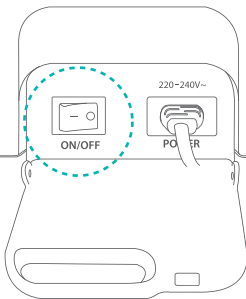


- 3** To restart the product, press the Start/Pause button and repeat steps 1 and 2.
- 4** When you have finished using the product, turn the power switch off.

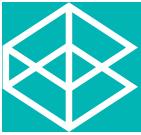


4. Auto Mode

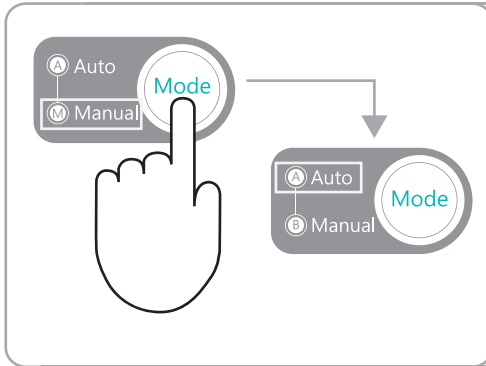


- 1** You can switch to Manual Mode during Auto Mode by pressing the Mode button.

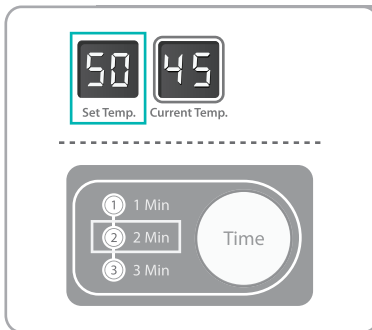



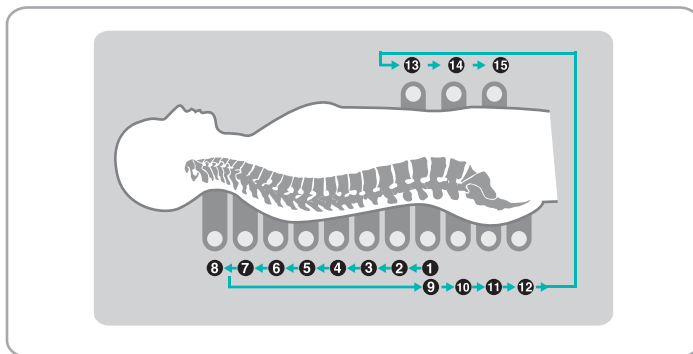


- 2** The projector will automatically revert back to the last set temperature and pressure time. The temperature can be set between 30~60°C and the pressure time can be set to 1, 2, or 3 minutes according to the user.

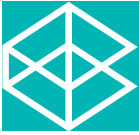

- 3** The light along the pressure points 1~15 will advance according to the set time.
- 4** You can switch to Manual Mode during Auto Mode by pressing the Mode button



5. Product Display Codes

Codes	Description
tH	 <ul style="list-style-type: none">■ The projector's temperature has exceeded the maximum setting temperature (60°C) by more than 10°C. The buzzer will go off twice every second for 5 seconds, and the product will automatically shut down■ Contact your local manufacturer's parts and service center for inspection or repair
-9	 <ul style="list-style-type: none">■ The projector's temperature is below -10°C■ The projector will begin to function properly as it heats up
SE	 <ul style="list-style-type: none">■ Internal sensor problem. Buzzer will go off twice every second for 5 seconds, and the projector will automatically shut off■ Contact your local manufacturer's parts and service center for inspection or repair
PH	 <ul style="list-style-type: none">■ The main body temperature is higher than 80°C. The buzzer will go off twice every second for 5 seconds, and the product will automatically shut down■ Contact your local manufacturer's parts and service center for inspection or repair
CH	 <ul style="list-style-type: none">■ The projector's electrical current has exceeded the normal amount. The buzzer will go off twice every second for 5 seconds, and the projector will automatically shut off■ Contact your local manufacturer's parts and service center for inspection or repair